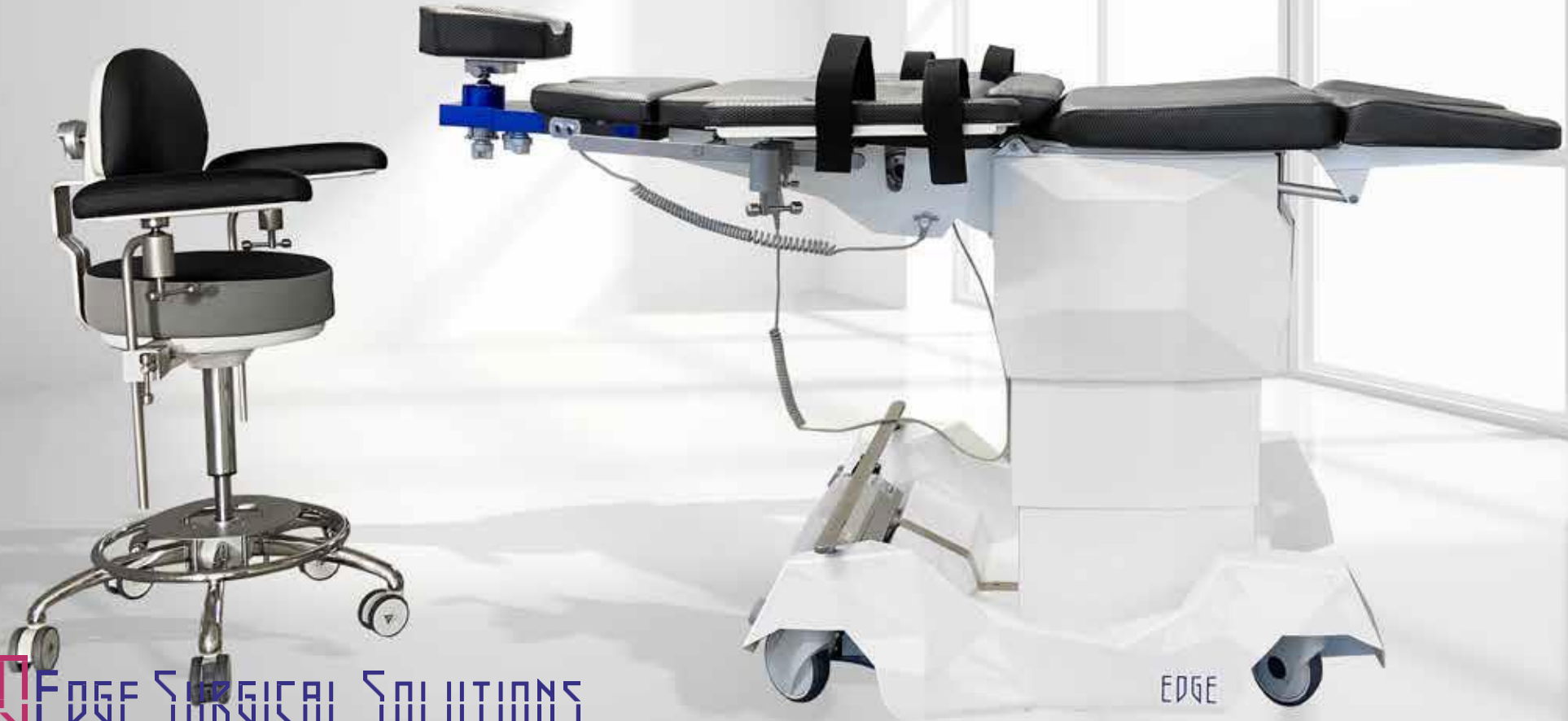
EDGE O TECH  
EDGE O EVO  
EDGE O

## Ophthalmic Surgery

 EDGE SURGICAL SOLUTIONS

EDGE

Registered Community Design

# WHO WE ARE | QUI NOUS SOMMES | CHI SIAMO

## THE COMPANY

EDGE SURGICAL SOLUTIONS is specialized in developing and manufacturing Multispecialty Examination and Surgery Chairs, Surgeon's Stools and Furniture.

EDGE SURGICAL SOLUTIONS is committed to the continuous research of innovative technical solutions for the Professionals; the range of equipment offered is based on and follows rigid guidelines: latest technology, perfect work-flow ergonomics, reliability, design and highly intuitive and easy to operate by the Professional.

The Company is directly involved through all development, manufacturing, assembly and quality control, shipment, installation and after-sale service phases thanks to its vast network of Distributors and Agents.

In the Company premises there is a permanent Show-Room and a Training Area for theory-practice sessions.

A well structured company organization that has enabled EDGE SURGICAL SOLUTIONS to grow at national and international level keeping the highest quality standards and constant innovation of its range of products.

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*“All EDGE SURGICAL SOLUTIONS Medical Equipment is designed, developed and manufactured in Italy”*

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# HOW WE WORK | COMMENT NOUS TRAVAILLONS | COME LAVORIAMO

## R&D

EDGE SURGICAL SOLUTIONS R&D Team develops internally all the equipment and devices for the Professionals and Clinics, always combining modern and essential design to the latest and most innovative technical solutions.

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R&D  
R&D  
R&D

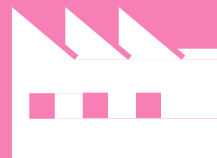
EDGE SURGICAL SOLUTIONS manufactures a wide and complete range of Surgical Chairs, Doctor Stools and Furniture. Every single production phase is planned and performed according to the most rigid quality control criteria.

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PRODUCTION  
PRODUCTION  
REALIZZAZIONE

EDGE SURGICAL SOLUTIONS will design your Ambulatory Surgery Center in order to allow for the best work-flow and ergonomics while guaranteeing maximum comfort to the Patient, according to the space availability and your different set up needs as a Professional.

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PROJECT  
PROJET  
PROGETTO

INSTALL  
INSTALL  
INSTALL

All EDGE SURGICAL SOLUTIONS equipment is "Plug-in & Work" easy to set up and ready to use with very few and easy installation steps. EDGE SURGICAL SOLUTIONS guarantees a highly professional installation service thanks to its vast network of Distributors and Agents, and a product specialist will train the Ambulatory staff on the correct use of the installed equipment.

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EDGE SURGICAL SOLUTIONS is committed to continuous training and information of all its Customers; the Company can rely on the cooperation with prominent experts in each medical discipline and often involved in training sessions on the specific equipment and different operating methods.

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EDGE SURGICAL SOLUTIONS considers Warranty a professional duty and mandatory towards all its Customers, reason being the Company extends guarantee of the equipment up to 2 Years on Electronic Parts and up to 5 Years on Mechanic Parts.

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All Customers can rely on a punctual and qualified after-sale assistance thanks to the in-house Service Department. You will be able to request a Preventive Maintenance and Courtesy Service to our Distributors and avoid any unexpected interruption of your professional activity.

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SERVICE  
SERVICE APRES-VENTE  
ASSISTENZA

# AMBULATORY SURGERY | CHIRURGIE AMBULATOIRE | CHIRURGIA AMBULATORIALE

## A Transformative Model for Modern Surgery and Procedures in ASC and Outpatient Rooms.

Ambulatory Surgery Centers—known as ASCs—are modern health care facilities focused on providing same-day surgical care, including diagnostic and preventive procedures.

ASCs have transformed the outpatient experience for millions of Patients around the World by providing them with a more convenient alternative to hospital-based outpatient procedures—and done so with a strong track record of quality care and positive patient outcomes while proving highly cost-effective.

Forty years ago, virtually all surgery was performed in hospitals; waits of weeks or months for an appointment were not uncommon, and

patients typically spent several days in the hospital and several weeks out of work in recovery. In many countries, surgery is still performed this way, but it is rapidly changing.

Physicians faced with frustrations like scheduling delays, limited operating room availability, slow operating room turnover times, and challenges in obtaining new equipment due to hospital budgets and policies, physicians were looking for a better way—and developed it in ASCs.

EDGE SURGICAL SOLUTIONS has fully committed to give a contribution to the development of technically advanced, elegant and cost-effective equipment that enables Physicians to perform Examinations and Procedures according to the time saving principle of rotation: prepare the Patient, perform Surgery and keep Patient for observation and recovery on the same Chair. When the Physician is

operating, staff can prepare the following Patient for surgery and at the same time the previous Patient is safely and comfortably recovering on the same chair. The Patient is never moved from the chair, access is easily granted thanks to the extremely low entry/exit height of the chair, and then by simply pressing the memory buttons the Patient will immediately reach operating position previously set and memorized by the Physician. The chair turns to an operating table, then to a recovery bed and back to examination chair by simply operating the wireless foot-control, and all of this performed at ease, with maximum comfort for the Patient and best ergonomics for the Physician.

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## Modern Surgery and Procedures in ASC and Outpatient Rooms

EASY ACCESS

PATIENT TRANSPORT

PREOPERATIVE PREPARATION

# EDGE TECH

Ophthalmic Surgery Chair

Is the brand new concept of Ophthalmic Surgery Chair developed and manufactured by EDGE SURGICAL SOLUTIONS.

It is the result of innovative technical engineering solutions allowing the Professional to perform any ophthalmic surgery and procedure, and also reducing operating times thanks to the "circular organization methodics" preparation of the Patient, surgery, observation and recovery period - always with the same Chair, avoiding the uncomfortable and time-consuming displacement of the Patient.

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EDGE Registered Community Design

# EDGE O TECH

Equipped with a section with an electric headrest. The whole mechanism allows any necessary micro-adjustments for the positioning of the patient's head.

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Equipped with 4 wheels wheels run by an electronic brake system. The ergonomics and safety of the wheel locking system improves the positioning and moving of the Chair.

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The solid metal structure, mobile base is equipped with 4 medical grade wheels run by an electronic brake system. The Chair is operated by 3 linear actuators and a lifting column by means of the multifunction upholstery includes 5 removable padded sections for easy cleaning and disinfection.

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It chair reaches a minimum height of 52 cm allowing patients with different medical conditions to easily access the chair from the front or the sides without any barrier.

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It can automatically reach the Operating Position thanks to the memorizable positions function activated by the foot control.

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It is equipped with a solid metal structure, whose mobile base is equipped with 4 medical grade wheels run by an electronic brake system, in order to provide maximum maneuverability.

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The chair can reach a perfect flat position at any height, for the maximum comfort of the Operator while performing a clinical examination or professional treatment. Maximum height 92 cm.

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Zero position of the chair.  
The chair automatically reaches "0" position to facilitate the access and the exit of the Patient.

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It allows the operator to perform "Synchro" movements of the Chair enables the combined and synchronized movement of the back-rest and leg-rest.

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It can rely on a lifting capacity of 325 kg, one of the highest in its category.

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It reaches Trendelenburg position with 15° inclination.

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The upper section of the backrest is made up of anodized aluminum elements to ensure maximum stability and lightness. The section is motorized via a linear actuator and stabilized by a gas spring kept under tension.

The headrest is shaped and padded according to the needs of different medical sectors and can be completely removed if necessary. The headrest is 3-way adjustable and can be safely locked when the correct position is achieved.

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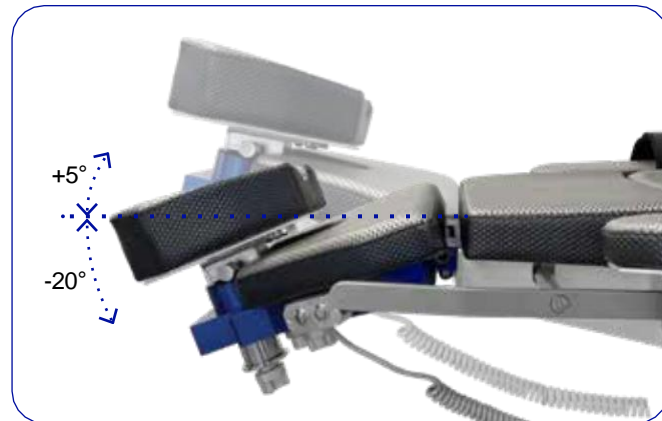
The headrest section can be adjusted from  $-20^{\circ}$  to  $+5^{\circ}$  by means of a 2-button stainless steel control panel. The control panel can be fixed on both sides of the section and hung with a quick magnetic coupling. **PROVIDED IN THE STANDARD VERSION**

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The headrest can be height-adjustable and can be locked in the desired position by means of its special clamping system.  
**PROVIDED IN THE STANDARD VERSION**

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The headrest section can be extended telescopically in order to fit different patient heights.

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The headrest can rotate 360° on a ball joint in order to achieve the most suitable position for the patient's treatment.

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Arm-rest is shaped and padded according to the requirements of different medical sectors and can all be completely removed when necessary.

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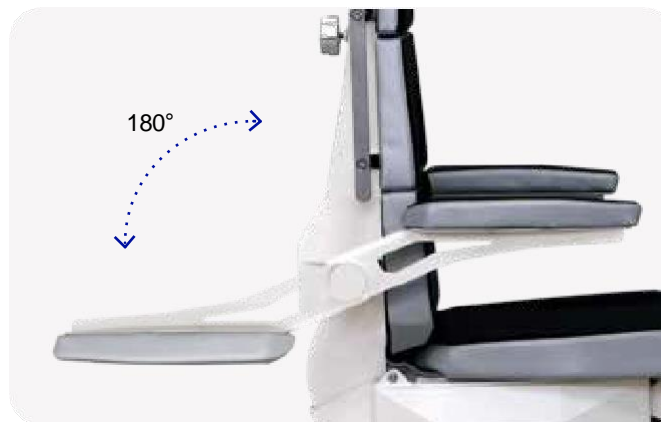
Both padded arm-rest are independently collapsible, removable and mechanically combined with the seat/back-rest movements ensuring comfort for the patient in any position.

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Padded and shaped leg-rest.  
**PROVIDED IN THE STANDARD VERSION**

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Leg-rest operated with a multifunction foot control by with a linear actuator.

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Stainless steel retractable foot-rest with a load capacity of 80Kg provides maximum comfort to the Patient.

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Stainless steel side rails mounted on the back-rest.

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Stainless steel side rails mounted on the SEAT.

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**EN OPTION**

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**OPZIONALE**

It is equipped with 4 wheels run by an electronic brake system.

The wheels can be locked and released easily and ergonomically by means of the key buttons situated on the back of the Chair. A wheel with directional lock allows the Professional to move the Chair over long distances with ease.

The motion sensor connected to the wheels allows the Chair to be automatically locked after 30 sec. of stop.

It is equipped with a 5.0 Ah battery allowing it to operate the

Chair autonomously, without connection to the power supply.

The battery allows 200 movements of the Chair. The charge indicator on the battery allows monitoring the charge level.

The battery recharges autonomously when the Chair is connected to the power supply.

## PROVIDED IN THE STANDARD VERSION

- Total or directional lock (0/180°)
- Housing and wheel centre glass fibre- reinforced polyamide
- Wheel tread polyurethane 80 shore A, hydrolysis resistant
- Ball bearing in swivel head and double row ball bearing in wheel
- Wheel diameter 125 mm
- Load capacity 150 kg

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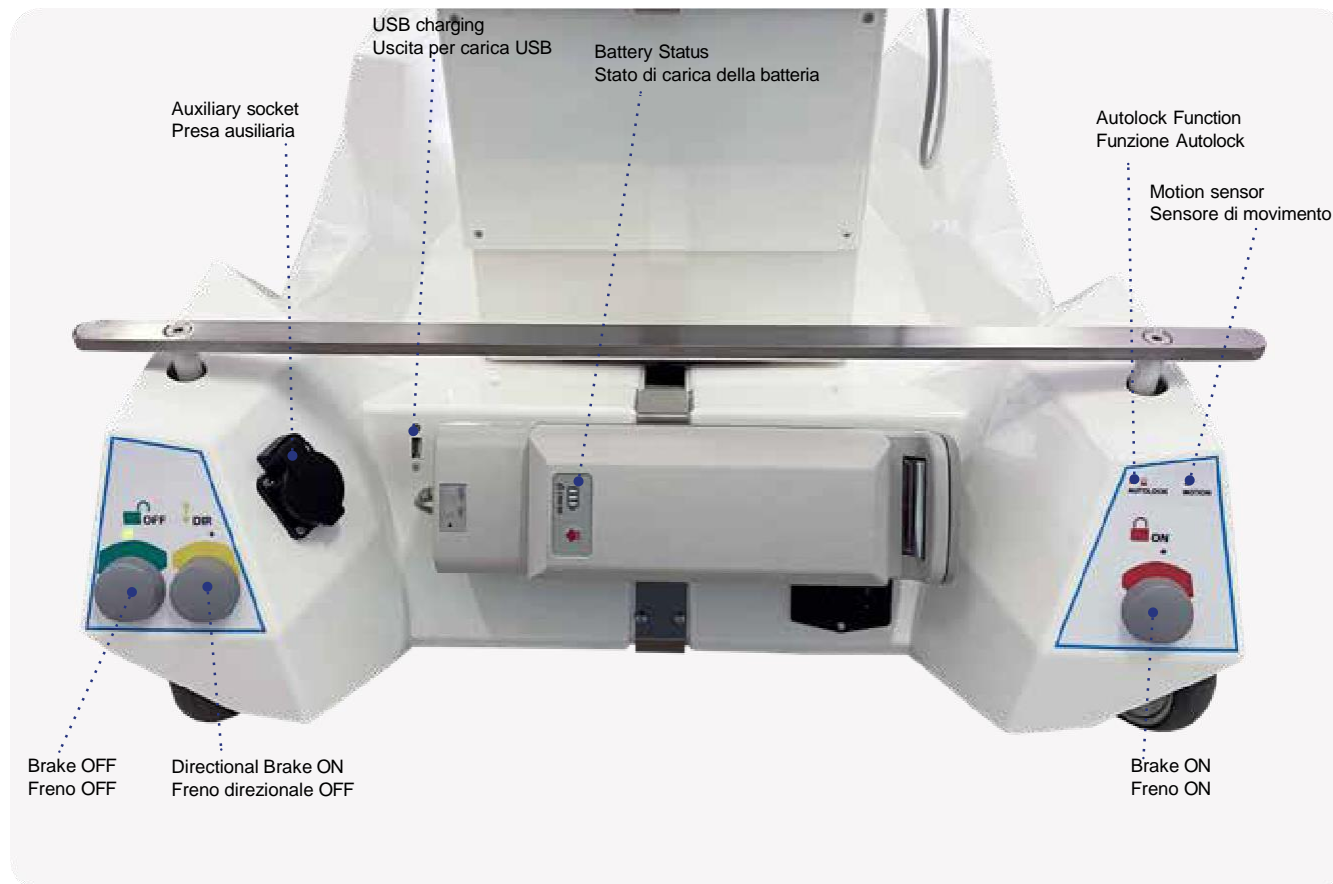
It is provided in the standard version with the structure in White color or Silver at choice.

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The upholstery comprises 5 separate padded sections fully detachable for easy cleaning and disinfection, made of high density undeformable foam and the cover is a bi-elastic, abrasion-resistant, flame retardant and antimicrobial medical grade Skai available in over 80 different colours and textures.

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# EDGE OTECH

EDGE OTECH is operated by a wireless foot control with modern design and extremely intuitive. Easy to operate and clean, the foot control provides the Professional with an ergonomic and efficient working tool, useful to position the Patient comfortably and memorize different working or safety positions.

The foot control has 8 buttons to operate the 4 actuator and move the Chair sections; it is also provided with 2 memorizable positions buttons (M1, M2), the easy Patient Access-Exit position (M0), and the Sync button that enables leg-rest and back-rest simultaneous movement.

The foot control is also equipped with two buttons for the emergency positions of the Chair: the flat position and the Trendelenburg position. The pedal innovative visual and acoustic signals communicate all safety, memorized movements and charging activity of the chair and it's foot control.

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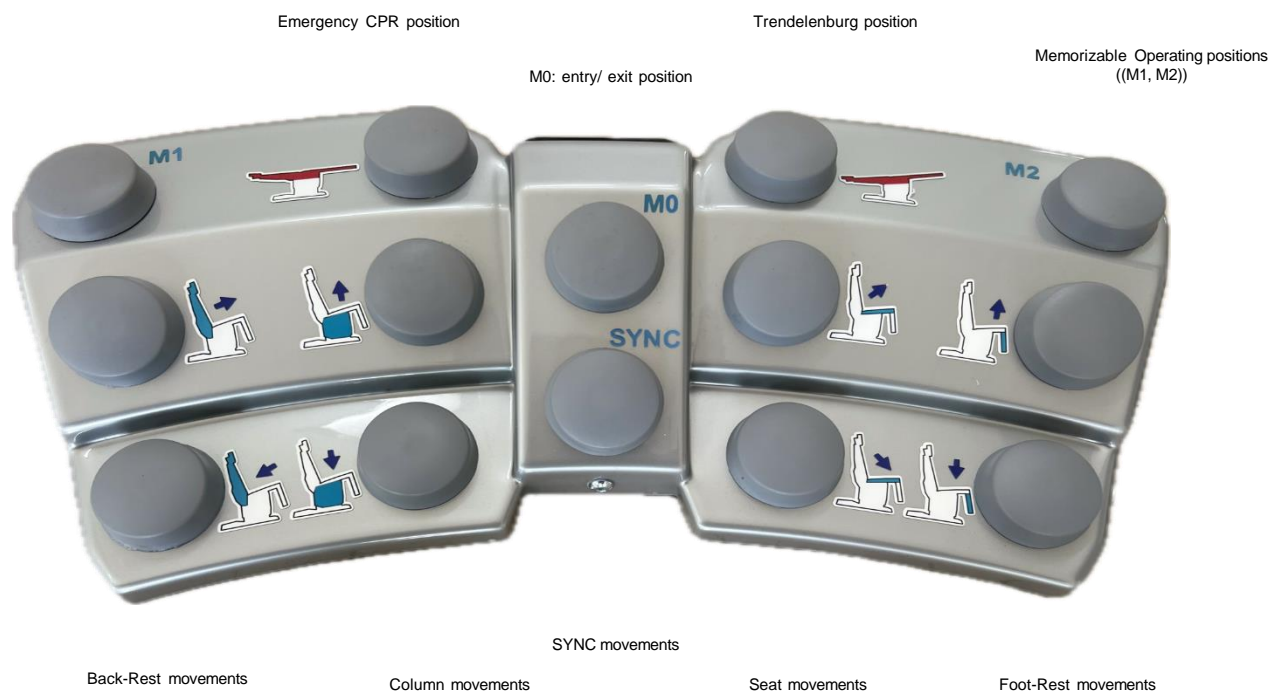
# EDGE O EVO

Ophthalmic Surgery Chair

EDGE O EVO is the brand new concept of Ophthalmic Surgery Chair developed and manufactured by EDGE SURGICAL SOLUTIONS.

It is the result of innovative technical and engineering solutions allowing the Professional to perform any ophthalmic surgery and procedure, and also reducing operating times thanks to the "circular organization methodics" - preparation of the Patient, surgery, observation and recovery period - always with the same Chair, avoiding the uncomfortable and time-consuming displacement of the Patient.

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Registered Community Design

# EDGE O EVO

It is equipped with 4 wheels run by an electronic brake system. The ergonomics and safety of the wheel locking system improves the positioning and moving of the Chair. The wheels can be locked and released easily and ergonomically by means of the key buttons situated on the back of the Chair. A wheel with directional lock allows the Professional to move the Chair over long distances with ease. The motion sensor connected to the wheels allows the Chair to be automatically locked after 30 sec. of stop.

Conceived with a Solid metal structure, whose mobile base is equipped with 4 medical grade wheels run by an electronic brake system. The Chair is operated by 3 linear actuators and a lifting column by means of the multifunction and wire-less foot control. The upholstery includes 5 fully removable padded sections for easy cleaning and disinfection, made of non-deformable high-density foam and with a bi-elastic, abrasion-resistant, fire-retardant and antimicrobial skai coating, available in 80 different colors and 8 different textures of fabric. The whole structure is covered with an innovative and modern design ABS casing. All surfaces allow easy cleaning and disinfection.

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The chair reaches a minimum height of 52 cm allowing patients with different medical conditions to easily access the chair from the front or the sides without any barrier.

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Head-rest is shaped and padded according to the requirements of different medical sectors and can all be completely removed when necessary.

The articulated head-rests are adjustable in 3 directions, can be locked firmly when reaching the correct position of the Patient.

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**OPTIONAL HEAD-REST  
MORE INFO ON PAGE 38,39**

The headrest can be positioned in line with the back-rest upholstery. The sliding rod of the headrest allows a height excursion of 30 cm.  
**PROVIDED IN THE STANDARD VERSION**

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# EDGE O EVO

EDGE O EVO is operated by a wireless foot control with modern design and extremely intuitive. Easy to operate and clean, the foot control provides the Professional with an ergonomic and efficient working tool, useful to position the Patient comfortably and memorize different working or safety positions.

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The foot control is also equipped with two buttons for the emergency positions of the Chair: the flat position and the Trendelenburg position. The pedal innovative visual and acoustic signals communicate all safety, memorized movements and charging activity of the chair and its foot control.

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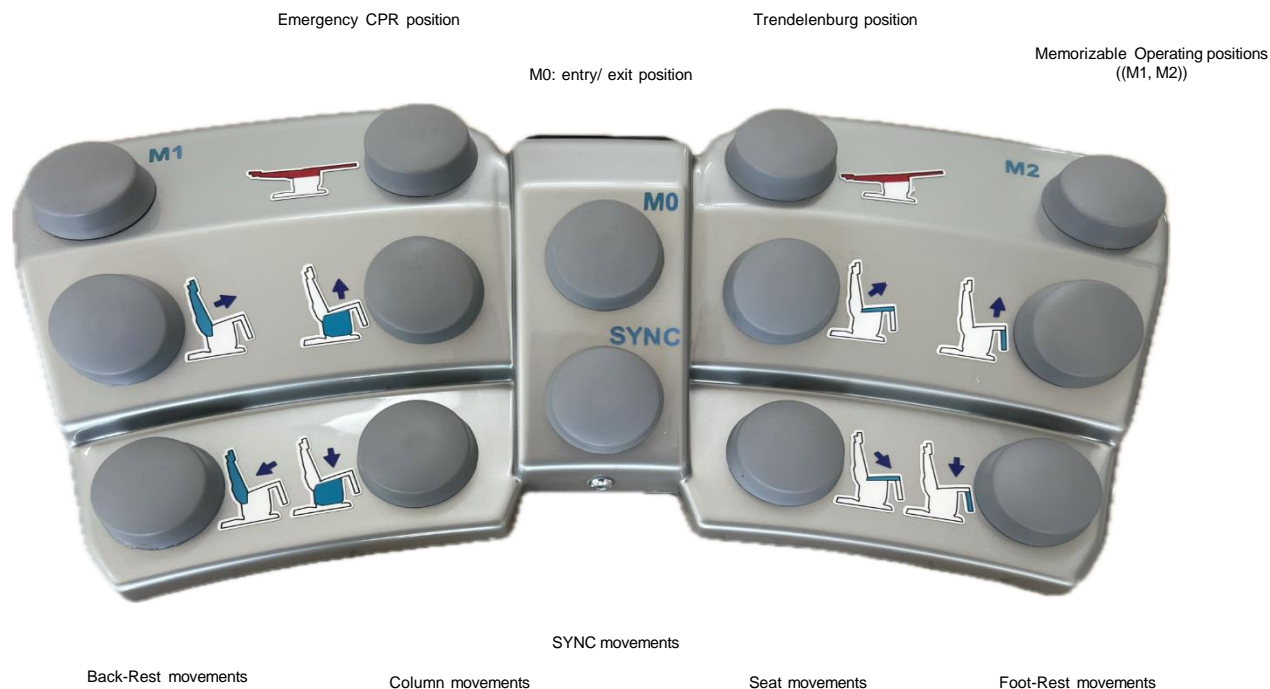
# EDGE<sup>o</sup>

Ophthalmic Surgery Chair

is EDGE SURGICAL SOLUTIONS ultimate concept of Mobile Ophthalmic Surgery Chair. It is the result of innovative technical and engineering solutions that enable the Professionals to perform Ophthalmic Surgery according to the time saving principle of rotation: prepare the patient, perform surgery and keep patient for observation and recovery. It is the perfect working tool that suits all Ophthalmology Clinics and Private Practices.

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EDGE  
Registered Community Design

# EDGE<sub>0</sub>

It is conceived in a solid metal structure, the mobile base is equipped with 4 medical grade castors and rear lock and It is operated with 3 silent linear actuators and one telescopic column by a modern and multi-function wire-less foot control. The upholstery comprises 5 separate padded sections fully removable for easy cleaning and disinfection, made of high density undeformable foam and the cover is an abrasion-resistant, flame retardant and antimicrobial Skai available in 80 different colours and 8 textures.

It is designed for the comfort of the patient and the ergonomic of the operator, it reaches a minimum height of 53 cm (21" inches) and a maximum Height of 93cm(37" inches) allowing patients with medical conditions to easily access the chair from the front or the sides without any barrier, while allowing the operator to position the chair in a multiple range of heights and inclinations enabling the perfect operating position.

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# EDGE<sub>0</sub>

53 cm

The chair reaches a minimum height of 53 cm allowing patients with different medical conditions to easily access the chair from the front or the sides without any barrier.

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castor lock

360°

It is equipped with medical grade castors, two of them are pivoting while 2 fixed, and central brake system at rear, in order to provide maximum maneuverability.

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93 cm

The chair can reach a perfect flat position at any height, for the maximum comfort of the Operator while performing a clinical examination or professional treatment. Maximum height 93 cm.

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# EDGE<sub>0</sub>

It is operated by a wireless foot control with modern design and extremely intuitive. Easy to operate and clean, the foot control provides the Professional with an ergonomic and efficient working tool, useful to position the Patient comfortably and memorize different working or safety positions.

The foot control has 8 buttons to operate the 4 actuator and move the Chair sections; it is also provided with 2 memorizable positions buttons (M1,M2), the easy Patient Access-Exit position (M0), and the Sync button that enables leg-rest and back-rest simultaneous movement.

The foot control is also equipped with two buttons for the emergency

positions of the Chair: the flat position and the Trendelenburg position. The pedal innovative visual and acoustic signals communicate all safety, memorized movements and charging activity of the chair and it's foot control.

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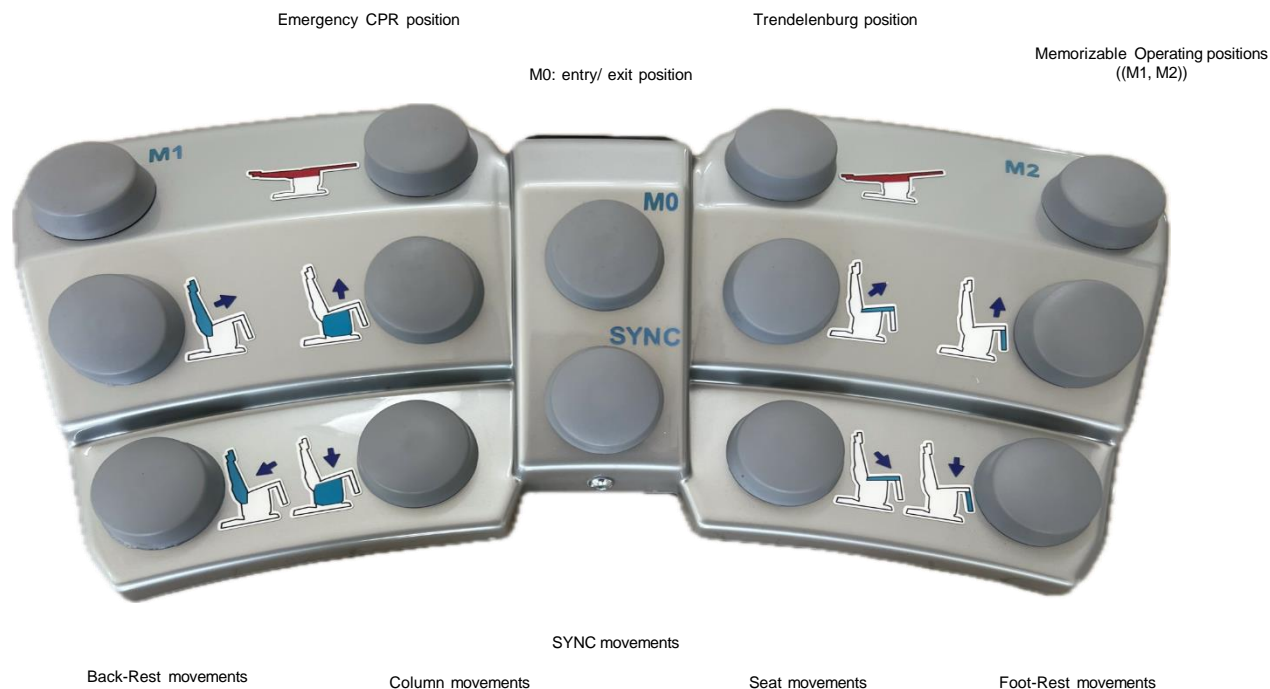
Triple Articulation Head-Rest for perfect positioning of the Patient's head in any required cranio-facial examination/surgery.

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SUITABLE FOR:

EDGE<sup>o</sup> EVO  
EDGE<sup>o</sup>

**STANDARD HEAD-REST  
MORE INFO ON PAGE 23, 33**

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Sculptured Head-rest. For high precision patient positioning and head fixation; adjustable manually by articulated mechanism. Ideal for Ophthalmology, ENT, Maxillofacial Surgery, Plastic and Aesthetic Surgery.

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SUITABLE FOR:

EDGE<sup>o</sup> EVO  
EDGE<sup>o</sup>

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Side Support, it is ideal for patient stabilization and containment; it is collapsable and removable. Easy to clean and disinfect.

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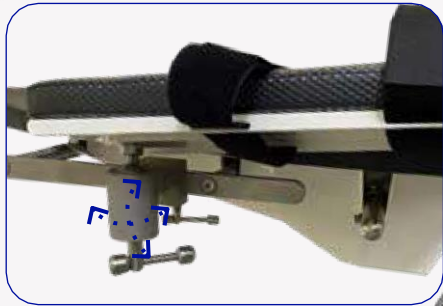
Articulated armrests on mechanical ball joint with removable upholstery and Velcro containing belt. Also suitable for Anesthesia and Blood Sampling.  
**OPTIONAL**

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Stainless steel tray support, extendable and height adjustable.  
**OPTIONAL**

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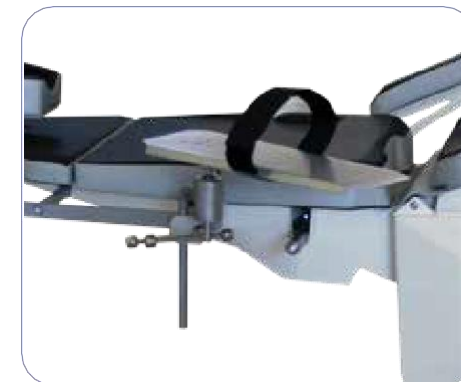
I.V Arm-rest, it rotates 360° and is swiveling and tiltable by means of ball joint.  
**OPTIONAL**

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| codice  | descrizione  |
|---------|--|
| 9110636 | FLEXIBLE<br>DRAPE<br>SUPPORT<br>WITH<br>OXYGEN<br>CONNECTION |

Flexible Surgical Drape Support  
with Oxygen Connection and Clamp  
Included  
**OPTIONAL**

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Flexible Surgical Drape Support with Clamp Included  
**OPTIONAL**

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Fore-Arms Support. The height-adjustable micro-surgery arm support, with its horse-shoe shape perfectly fits the head-rest region, providing maximum comfort to the Surgeon. It is adjustable in height and orientation, it is padded for maximum comfort to the fore-arms and guarantees ideal ergonomics during the procedures.

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The Hand Control enables the required positioning of the Patient, by activating the linear actuators that control the position of each independent section (lifting column, seat, back-rest and foot-rest).

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**OPZIONALE**

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Additional battery (5Ah capacity), equipped with capacity indicator and external charging station.

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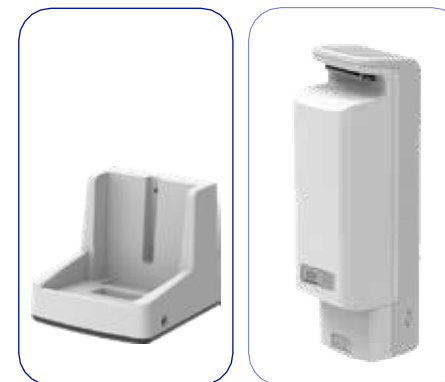
EDGE<sup>o</sup> TECH  
EDGE<sup>o</sup> EVO

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SUITABLE FOR:  
**EDGE 0**

Battery kit including two batteries (capacity 5 Ah), with capacity indicator and recharger, allowing to operate the Chair without electrical supply connection. Replacing the battery with the spare one provided is very simple.

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SUITABLE FOR:  
**EDGE 0**

Back up battery with 2Ah capacity allowing to operate the Chair in the event of a temporary power outage.

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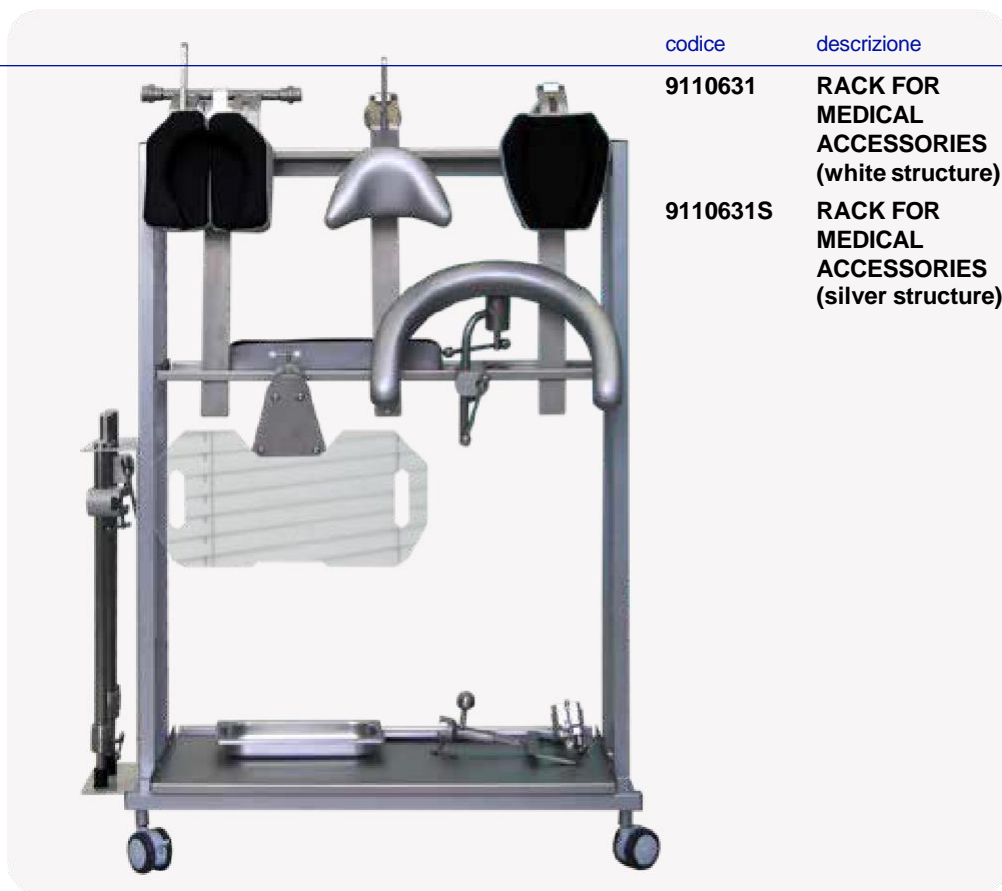
| codice   | descrizione                                     |
|----------|---|
| 9110631  | RACK FOR MEDICAL ACCESSORIES (white structure)  |
| 9110631S | RACK FOR MEDICAL ACCESSORIES (silver structure) |

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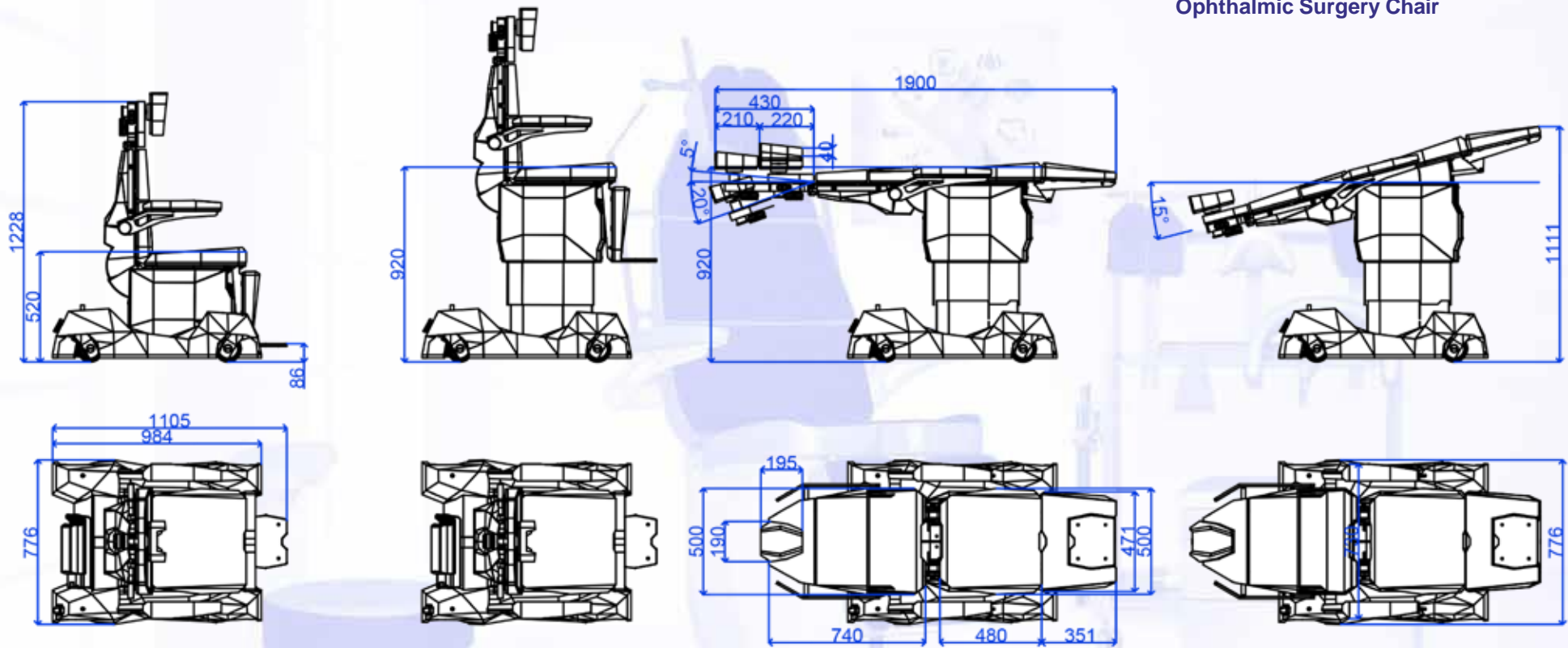
**EDGE O TECH**  
 Ophthalmic Surgery Chair

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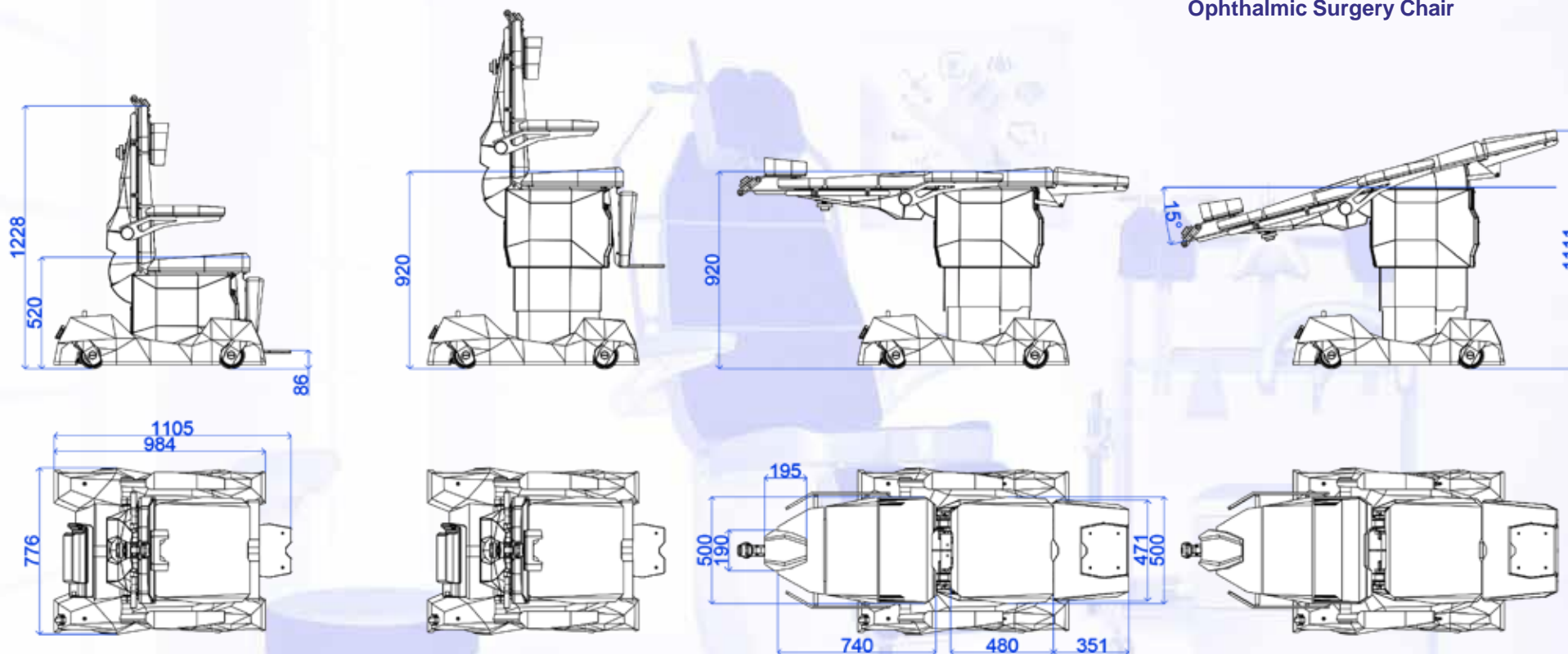
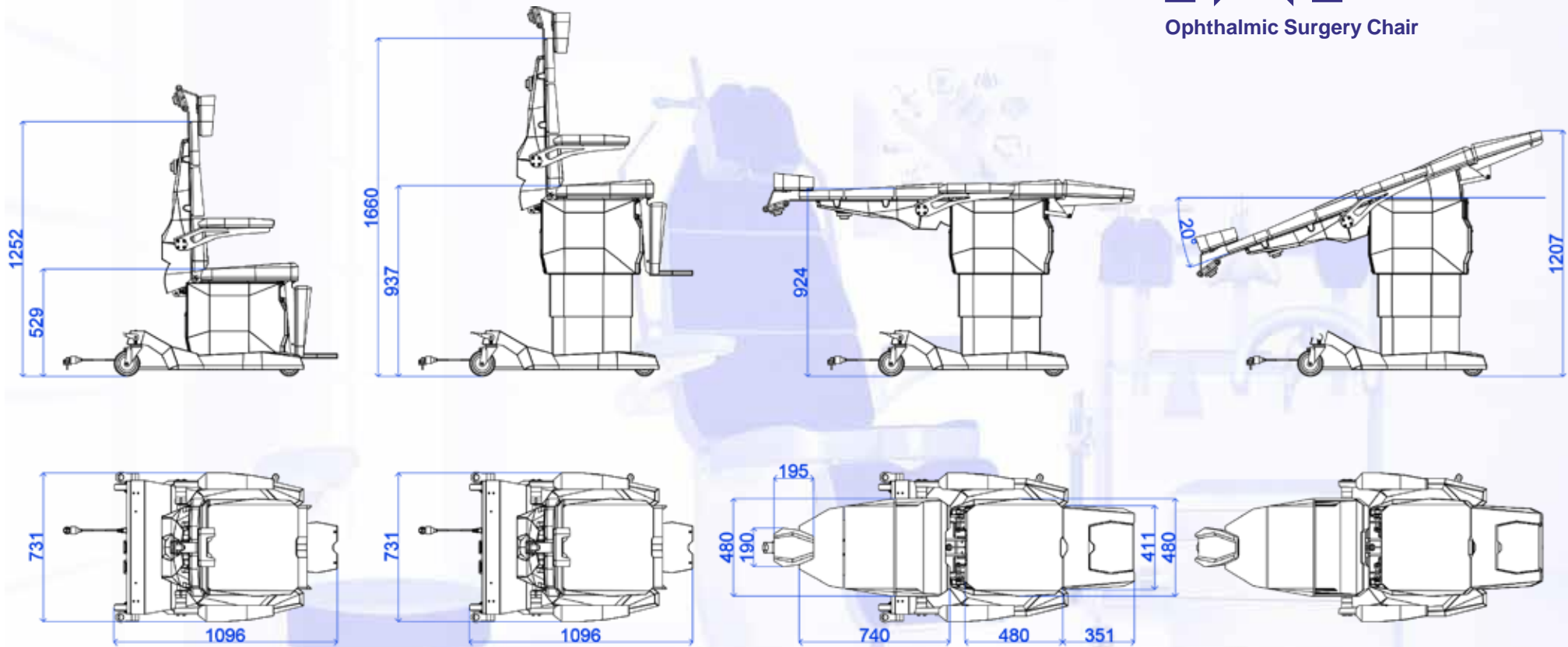
Doctor stool with anatomical seat. Adjustable height of the backrest. Inclination angle of the backrest is adjustable in a fixed position or can dynamically support the back muscles. Control lever for all adjustments. Base in chromed steel. Seat height adjustable by gas spring. High quality skay coating.

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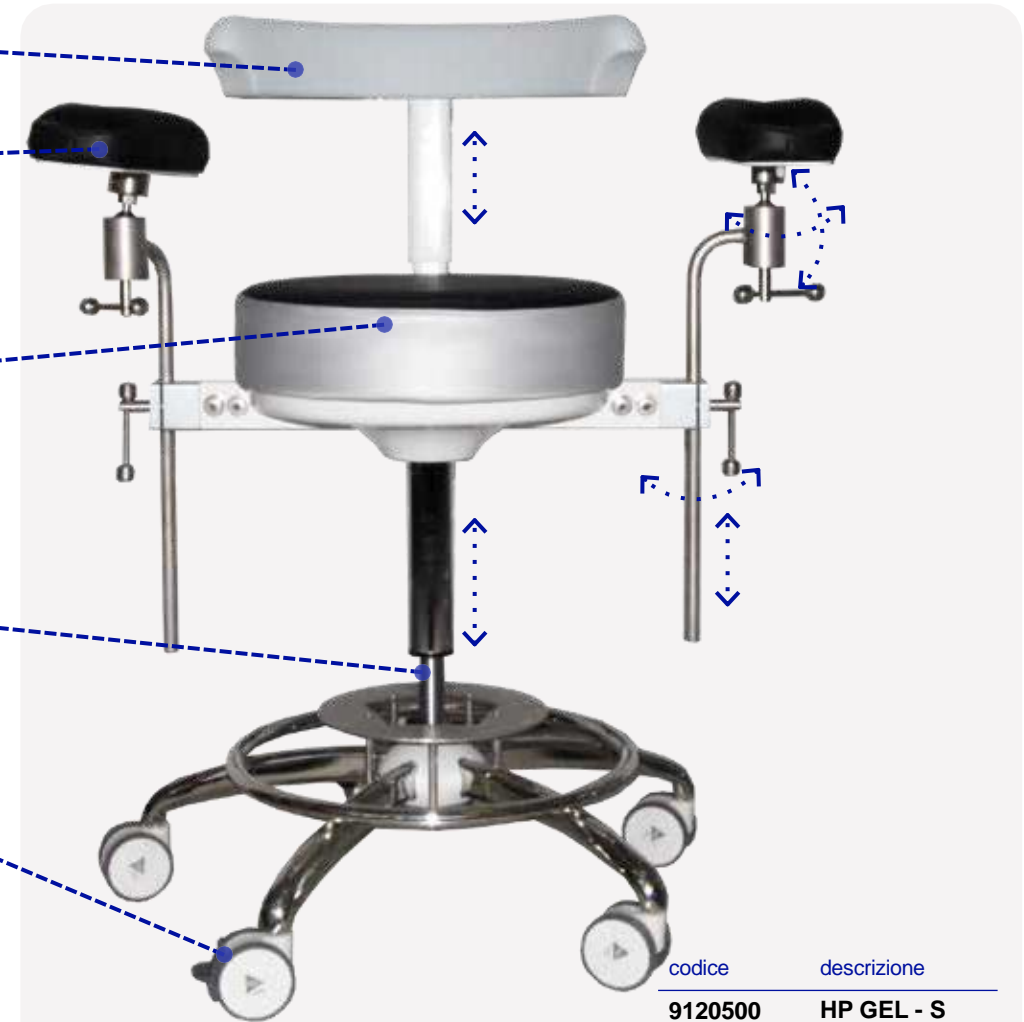
### OPZIONALE

80 colours available

| codice  | descrizione                            |
|---------|--|
| 9120500 | HP GEL - S<br>(without foot-rest)      |
| 9120506 | HP GEL - S<br>(with foot-rest)         |
| 9120502 | HP GEL - S HIGH<br>(without foot-rest) |
| 9120507 | HP GEL - S HIGH<br>(with foot-rest)    |

80 colours available

| codice  | descrizione                              |
|---------|--|
| 9120133 | HP GEL<br>WITHOUT<br>FOOT REST           |
| 9120134 | HP GEL WITH<br>FOOT REST                 |
| 9120136 | HP GEL<br>WITHOUT<br>FOOT REST -<br>HIGH |
| 9120137 | HP GEL WITH<br>FOOT REST -<br>HIGH       |

The seat naturally adapts to the user's body by increasing the contact area and decreasing the pressure on it. Circular seat with visco-elastic foam in different densities and memory effect. Multifunction armrest/back rest in high resistant polyurethane. Base in steel and polished chrome. 5 self braking wheels. Adjustable height by gas piston. High quality skay cover.

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**OPZIONALE**

EDGE SURGICAL SOLUTIONS develops and manufactures cabinets and furniture for: Ambulatory Surgery Center, Sterilisation Room, Outpatient Room and Front Office. The EDGE SURGICAL SOLUTIONS Furniture Project is paramount thanks to its innovative constructive and installation solutions that allow for an easier operating activity in the room, speed up the picking of needed instruments and material, guarantee the use of single modules as workstations during the procedures reducing obstacles and inefficiencies while allowing for a faster and easier disinfection of the operating area at the end of the activity when all modules can be stored back in the Corian® Garage. It's a space saving, clean and elegant setup that can be adapted to any room size.

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EDGE SURGICAL SOLUTIONS cabinets are entirely built in metal sheet, riveted and painted with epoxy powder, the drawers modules are independent and mobile thanks to the 4 medical grade castors with 2 front brakes and no-mark rubber; the drawers are finished with metal front panels, different dimensions and colours at choice, push-pull opening/closing system and handles/free for a better and easier disinfection. The Sink is built in the Corian® working top and it guarantees perfect disinfection. State of the art design, elegant and high quality material, perfect ergonomics and absolute infection control define this EDGE SURGICAL SOLUTIONS unique project as a benchmark in Medical Furniture.

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Garage Structure in steel and completely covered in Corian® to house the mobile modules. Splash-proof Corian® worktop guarantees easy cleaning and disinfection and long durability.

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Drawers with push-pull system with stabilizer bar. Internal abs instruments trays for perfect disinfection and perfect storage.

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Mobile Cabinet Modules, 4 medical-grade, non-marking wheels equipped with two front brakes for easy maneuverability.

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Corian® Worktop for an easier maneuverability of the mobile modules.

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Sink integrated on the Corian® worktop, available in both stainless steel and Corian® in different sizes and models.

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Hanging cabinets with drawers and flaps in different sizes and wall furniture with custom-made shelves.

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Possibility to integrate an archive and reporting desk with Methacrylate divider from operating area.

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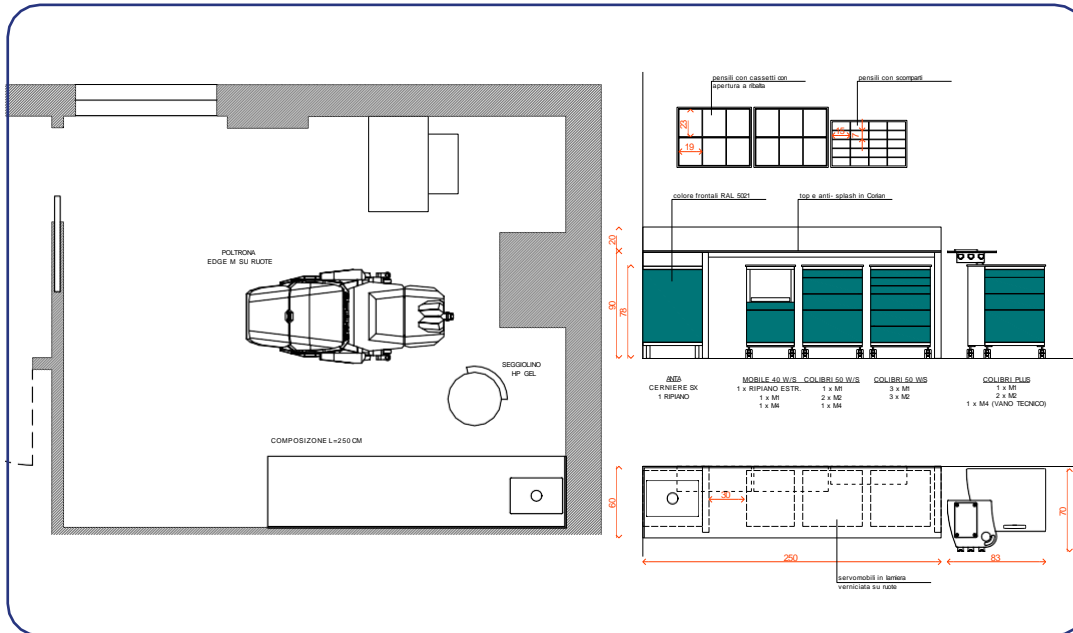
Possibility of integrating a stainless steel insert on the Corian® worktop to place the hot trays taken out from the autoclave.

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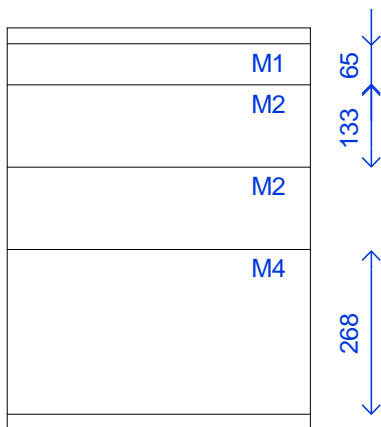
**Corian® - Technical Sheet**

- Corian® is a non-porous and homogeneous material, composed of approximately 1/3 acrylic resin and approximately 2/3 natural mineral substances.
- The junction points are imperceptible.
- Corian® surfaces are renewable (it can be fully restored to its original appearance by means of common detergents and an abrasive sponge).
- Corian® surfaces are hygienic.
- Corian® is an inert and non-toxic material.
- Corian® meets or exceeds emission guidelines for Volatile Organic Compounds (COV), Hazardous Air Pollutants (HAP).
- Chemical resistance. Class 1 reagents do not alter the material.

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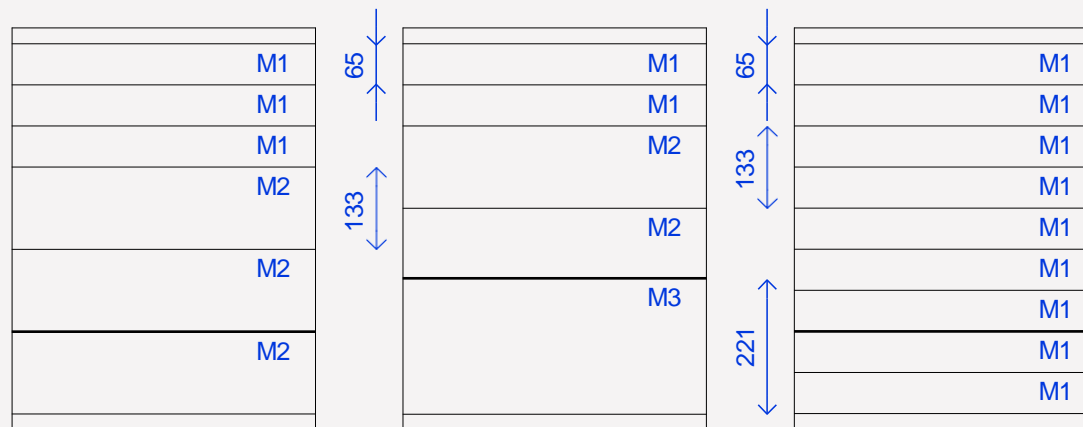
50 cm wide Cabinet with 4 drawers and metal fronts without handles provided with a Push-Pull opening system. Inside the drawers, storage trays for instruments. Cabinet on castors (front wheels equipped with brake) Worktop in Corian®.

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Cabinet 50 cm width. Standard configuration with 4 drawers with metal fronts without handles and Push-Pull opening system. Equipped with internal instrument trays. Corian® side compartment for the equipment lodging (load capacity 8 kg). Mobile on wheels for a comfortable handling (the two front wheels are equipped with brakes). Corian® worktop.

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Cabinet 85 cm width with 8 drawers (2 M1, 4 M2 e 2 M4) with metal fronts without handles and Push-Pull opening system. Equipped with internal instrument trays. Mobile on wheels for a comfortable handling (the two front wheels are equipped with brakes). Corian® worktop.

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Cabinet 90 cm width with 6 drawers (2 drawers 40 cm wide and 4 drawers 50 cm wide) with metal fronts without handles and Push-Pull opening system. Extendable metal shelf with Soft Touch system. Equipped with internal instrument trays. Mobile on wheels for a comfortable handling (the two front wheels are equipped with brakes). Corian® worktop.

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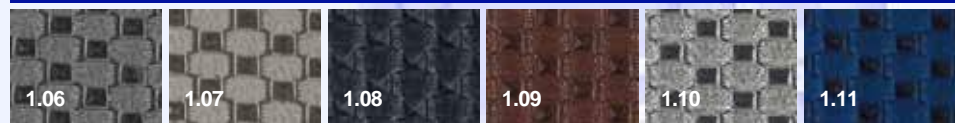
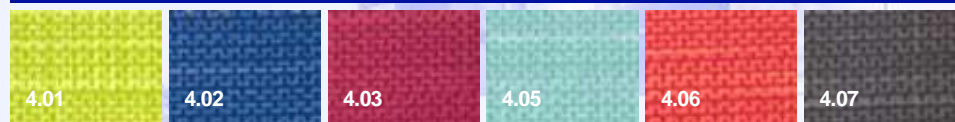
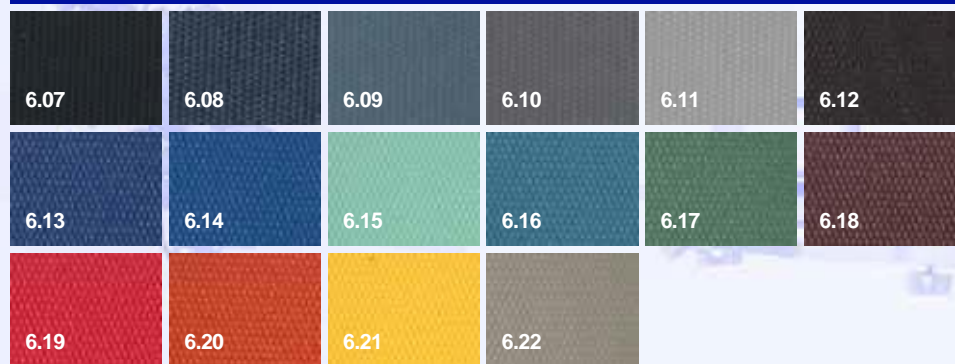
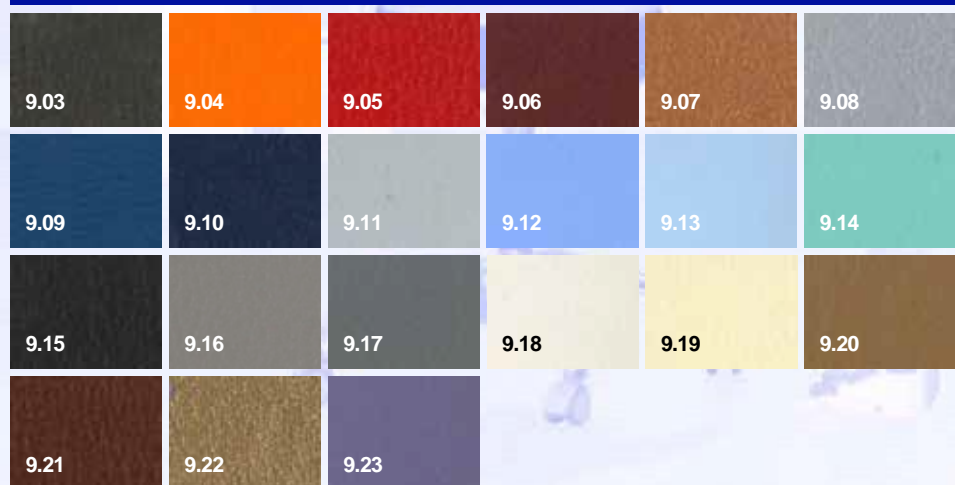
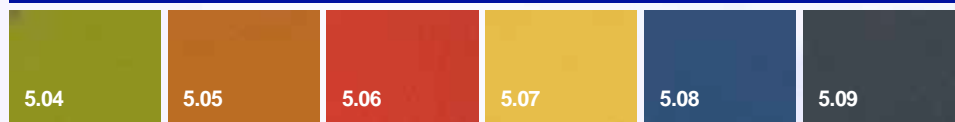
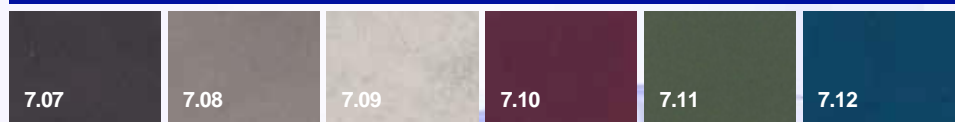
# UPHOLSTERY - SELLERIE - TAPPEZZERIA

## EDGE SURGICAL SOLUTIONS Cool

- Fireproof  
EN 1021.1-2  
IMO MSC 88/26/ADD.2 PART 8  
(3.1&3.2), ISO 3795
- Bleach resistant
- REACH Compliant  
Non toxic  
Non heavy metals  
No mercury
- No Bisphenol A (BPA)
- RoHS Compliant
- No PBTs (Persistent Bio accumulative and Toxic Chemicals)
- No significant colour change or stiffening  
UV coef  $\geq 7$
- Bromine-free, no Halogenated Flame Retardants (HFRs)
- Abrasion resistant  
100.000 cycles
- Phtalates free
- Mildew resistant

## EDGE SURGICAL SOLUTIONS Trend

- Fireproof  
EN 1021.1-2  
IMO MSC 88/26/ADD.2 PART 8  
(3.1&3.2)
- Bleach resistant
- REACH Compliant  
Non toxic  
Non heavy metals  
No mercury
- No Bisphenol A (BPA)
- RoHS Compliant
- No PBTs (Persistent Bio accumulative and Toxic Chemicals)
- No significant colour change or stiffening  
UV coef  $\geq 7$
- Bromine-free, no Halogenated Flame Retardants (HFRs)
- Abrasion resistant  
100.000 cycles
- Fire resistant additives free  
Antimicrobial resistant additive free  
Phtalates free

## EDGE SURGICAL SOLUTIONS Classic

- Fireproof  
M1 (NFP 92503)  
BS 5852 crib 5  
B2 (DIN 4102)  
EN 1021.1-2  
IMO A652  
FAR 25853 App F Part1 v12  
CATB 117 E  
CATB 133  
UFAC CLASS 1  
NY/NJ-Port Authority  
AM 18 (NFD 60-013)
- Waterproof
- Efficacy against bacteria (MRSA)
- Abrasion resistant  
ISO 5470-2  
150.000 cycles (12 KPa)
- Colour fastness to artificial light  $\geq 6$
- Phtalates free

## EDGE SURGICAL SOLUTIONS Glamour

- Fireproof  
EN 1021.1-2  
IMO MSC 88/26/ADD.2 PART 8  
(3.1&3.2), ISO 3795
- Bleach resistant
- REACH Compliant  
Non toxic  
Non heavy metals  
No mercury
- No Bisphenol A (BPA)
- RoHS Compliant
- No PBTs (Persistent Bio accumulative and Toxic Chemicals)
- No significant colour change or stiffening  
UV coef  $\geq 7$
- Bromine-free, no Halogenated Flame Retardants (HFRs)
- Abrasion resistant  
100.000 cycles
- Phtalates free
- Mildew resistant

## EDGE SURGICAL SOLUTIONS Brio

- Fireproof  
BS EN 1021-1 and 2  
NF P 92-503 (M2)  
IMO MSC 88/26/ADD.2 PART 8  
(3.1&3.2)  
CA TB 117-2013  
UFAC CLASS 1
- Bleach resistant
- REACH Compliant  
Non toxic  
Non heavy metals  
No mercury
- No Bisphenol A (BPA)
- RoHS Compliant
- No PBTs (Persistent Bio accumulative and Toxic Chemicals)
- No significant colour change or stiffening  
UV coef  $\geq 7$
- Bromine-free, no Halogenated Flame Retardants (HFRs)
- Abrasion resistant  
300.000 cycles
- Phtalates free
- Anti-microbial Mildew resistant

## EDGE SURGICAL SOLUTIONS Voyager

- Fireproof  
CL 1/ IM-M2-CRIB 5
- Waterproof
- UV resistant, low temp. resistant
- Mildew resistant, antibacterial and antimicrobial
- Free of heavy metal, formaldehyde, mercury and solvents
- Abrasion resistant  
ISO 5470-2  
100.000 cycles
- Chemical resistant to acid, bases, alcohol and common solvent

## EDGE SURGICAL SOLUTIONS Urban

- Fireproof  
M2 (NFP 92503)  
BS 5852 crib 5  
B2 (DIN 4102)  
EN 1021.1-2  
IMO A652  
FAR 25853 App F Part1 v12  
CATB 117 E  
CATB 133  
UFAC CLASS 1  
NY/NJ-Port Authority  
AM 18 (NFD 60-013)
- Waterproof
- Efficacy against bacteria (MRSA)
- Abrasion resistant  
ISO 5470-2  
120.000 cycles (12 KPa)
- Colour fastness to artificial light  $\geq 6$
- Phtalates free

## EDGE SURGICAL SOLUTIONS Premium

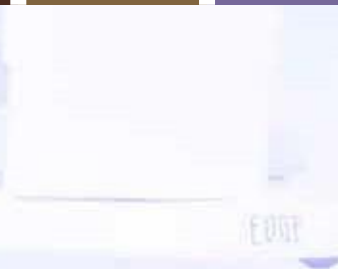
- Fireproof  
M2 (NFP 92503)  
BS 5852 crib 5  
B2 (DIN 4102)  
EN 1021.1-2  
IMO A652  
FAR 25853 App F Part1 v12  
CATB 117 E  
CATB 133  
UFAC CLASS 1  
NY/NJ-Port Authority  
AM 18 (NFD 60-013)
- Waterproof
- Efficacy against bacteria (MRSA)
- Abrasion resistant  
ISO 5470-2  
80.000 cycles (12 KPa)
- Colour fastness to artificial light  $\geq 6$
- Phtalates free

## EDGE SURGICAL SOLUTIONS Antistatic

- Electrically conductive (antistatic) synthetic leather
- Fireproof  
GOST
- REACH Compliant
- No significant colour change or stiffening  $\geq 6$
- Electric Resistivity  
 $10^4-10^6 \Omega$
- Phtalates free

## FRONT DRAWERS - TIROIRS AVANT - FRONTALI CASSETTI

|                    |                    |                        |                    |                    |                        |                    |                    |                        |                     |
|--------------------|--------------------|------------------------|--------------------|--------------------|------------------------|--------------------|--------------------|------------------------|---------------------|
| RAL 9023<br>(1.06) | RAL 1035<br>(1.07) | RAL 9023<br>(1.08)     | RAL 8029<br>(1.09) | RAL 9007<br>(1.10) | RAL 5026<br>(1.11)     | RAL 7042<br>(2.05) | RAL 7015<br>(2.06) | RAL 4002<br>(2.07)     | RAL 5013<br>(2.08)  |
| RAL 1001<br>(2.09) | RAL 8028<br>(2.10) | PANTONE 389C<br>(4.01) | RAL 5005<br>(4.02) | RAL 4010<br>(4.03) | RAL 6034<br>(4.05)     | RAL 3017<br>(4.06) | RAL 7036<br>(4.07) | PANTONE 391C<br>(5.04) | RAL 8023<br>(5.05)  |
| RAL 3031<br>(5.06) | RAL 1002<br>(5.07) | RAL 5023<br>(5.08)     | RAL 7024<br>(5.09) | RAL 7021<br>(6.07) | RAL 5008<br>(6.08)     | RAL 5009<br>(6.09) | RAL 7037<br>(6.10) | RAL 7036<br>(6.11)     | RAL 8019<br>(6.12)  |
| RAL 5000<br>(6.13) | RAL 5007<br>(6.14) | RAL 6027<br>(6.15)     | RAL 5025<br>(6.16) | RAL 6000<br>(6.17) | RAL 3005<br>(6.18)     | RAL 3027<br>(6.19) | RAL 3022<br>(6.20) | RAL 1032<br>(6.21)     | RAL 1019<br>(6.22)  |
| RAL 8017<br>(7.07) | RAL 7006<br>(7.08) | RAL 7044<br>(7.09)     | RAL 3005<br>(7.10) | RAL 6003<br>(7.11) | RAL 5009<br>(7.12)     | RAL 5022<br>(8.30) | RAL 3003<br>(8.31) | RAL 7040<br>(8.32)     | RAL 9010<br>(8.33)  |
| RAL 7036<br>(8.34) | RAL 8025<br>(8.35) | RAL 8017<br>(8.36)     | RAL 5009<br>(8.37) | RAL 6034<br>(8.38) | PANTONE 391C<br>(8.39) | RAL 1023<br>(8.40) | RAL 4010<br>(8.41) | RAL 8019<br>(9.03)     | RAL 2003<br>(9.04)  |
| RAL 3020<br>(9.05) | RAL 3005<br>(9.06) | RAL 8004<br>(9.07)     | RAL 9006<br>(9.08) | RAL 5007<br>(9.09) | RAL 5013<br>(9.10)     | RAL 7035<br>(9.11) | RAL 5023<br>(9.12) | RAL 5024<br>(9.13)     | RAL 6027<br>(9.14)  |
| RAL 7021<br>(9.15) | RAL 7039<br>(9.16) | RAL 7037<br>(9.17)     | RAL 1013<br>(9.18) | RAL 1015<br>(9.19) | RAL 8024<br>(9.20)     | RAL 8016<br>(9.21) | RAL 1036<br>(9.22) | RAL 4005<br>(9.23)     | RAL 7021<br>(10.01) |

[www.edgesurgicalsolutions.com](http://www.edgesurgicalsolutions.com)

[edgesurgicalsolutions@gmail.com](mailto:edgesurgicalsolutions@gmail.com)

CE

*“All EDGE SURGICAL SOLUTIONS Medical Equipment is designed ,  
developed and manufactured in Italy”*

## **EDGE SURGICAL SOLUTIONS Srl**

Headquarter: Via Umberto I, 48 73039 Tricase (LE) –ITALY

Production Site: S.S.275 Alessano-Lucugnano KM 17,5  
73031 ALESSANO (LE) - ITALY